



How to wear  
**BARRIER®** filtering half mask

## Donning of Barrier® filtering half mask



1. Separate upper and lower edges of the facemask.



2. Cup the mask under your chin and pull the headbands over the head.



3. Position the top band above the ears and the lower band below the ears and the base of the neck.



4. Conform the nose-clip across the nose with both hands and secure a tight seal on the face. Adjust both mask panels for a comfortable and safe fit. Perform user seal check (see below) and readjust mask if necessary.

## Doffing of Barrier® filtering half mask



1. Grasp the headbands and pull over the head.



2. Dispose the face mask directly holding the mask away from body by the headbands only. Disinfect your hands.

## User seal check

The seal check should be completed each time a new filtering half mask is donned. Exhale sharply while wearing the mask. If you feel air escaping around the nose or along the chin, re-adjust the nose clip to conform to the shape of the face. If air escapes near the mask edges, try to work the straps back or adjust the mask for better fit. If there is no proper fit, DO NOT enter the work area.

### If air leaks around the nose

Use both hands to readjust the nose piece by placing your fingertips at the top of the metal nose clip. Slide your fingertips down both sides of the metal strip to more efficiently mold the nose area to the shape of your nose.

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00  
The Mölnlycke and BARRIER trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. ©2020 Mölnlycke Health Care AB. All rights reserved. HQIM001811



**Mölnlycke**<sup>®</sup>