Mölnlycke[®] Melgisorb[®] Plus Melgisorb[®] Plus Cavity

Single use
Usage unique

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Melgisorb® Plus / Melgisorb® Plus Cavity

Absorbent Alginate Dressing / Absorbent Alginate Cavity Dressing

Product description

Melgisorb Plus is a soft, highly absorbent, sterile, single use dressing made of Calcium Sodium Alginate fibres. The alginate is hydrophilic and absorbs wound exudate. When absorbing exudate containing sodium ions, the solid alginate fibres quickly form a moist gel, which creates a moist environment beneficial for wound healing. When in contact with blood, Melgisorb Plus accelerates platelet aggregation, enhancing the haemostatic process. The gel can be easily removed from the fragile wound bed by irrigating with 0.9% saline.

Melgisorb Plus is available as flat and as cavity dressings.

Indications for use

Melgisorb Plus is intended for a wide range of moderately to heavily exuding wounds, both infected and non-infected, such as: pressure sores, venous and arterial ulcers, diabetic ulcers, post-operative wounds, dermal lesions and other external wounds inflicted by trauma.

Precautions

Melgisorb Plus is not indicated for dry wounds, third-degree burns or surgical implantation.

- Melgisorb Plus should not be used on heavily bleeding wounds.
- Do not use Melgisorb Plus on patient and/or user with known hypersensitivity to the ingoing materials/components of the product.
- If infection is suspected, follow local routines given by health care professionals.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.

Instructions for use

Melgisorb Plus can be used by lay persons under supervision of health care professionals.

Application:

1. Cleanse/flush wound with 0.9% saline or appropriate solution.
2. Dry the healthy surrounding skin and if needed, protect with a water-resistant cream/ointment.
3. Apply a dry Melgisorb Plus product to the moist wound bed.
Shallow wounds: Choose the correct size of flat dressing to be able to cover the entire wound.
Deep wounds and cavities: Cut the appropriate length of the cavity dressing and pack the wound loosely.
4. Cover with an appropriate secondary dressing. The choice depends on the exudate level. Secure with an adhesive dressing or a bandage as appropriate.
5. Compression therapy may be used in conjunction with Melgisorb Plus.

Dressing change and removal:

The dressing change interval may be several days. Change the dressing before it is fully saturated, or as indicated by clinical practice.

Remove Melgisorb Plus by gently cleansing/flushing with 0.9% saline or appropriate solution. Any non-gelled Melgisorb Plus will moisten in contact with the saline.

Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Melgisorb Plus it should be reported to Mölnlycke Health Care and to your local competent authority.

Melgisorb® Plus and Melgisorb® Plus Cavity are registered trademarks of Mölnlycke Health Care AB.

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