Avvertenze

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it

Application

1. Open the packaging and remove the dressing.
2. Mepiform can be cut to the required size but must overlap the scar by at least 1 cm.
3. Before applying Mepiform, make sure the area is dry. When used together with ointment or cream, ensure the dressing covers an area beyond the cream for best fixation.
4. Remove the release film and apply Mepiform over the closed wound/scar. Avoid stretching when applying over joints.

Changing or removing the dressing

- Mepiform should optimally be worn for 24 hours a day. Remove the dressing once a day for inspection and, if needed, wash the skin. The same dressing can then be reapplied to the same scar as this is a single patient multiple use product.
- Mepiform can be used for up to 7 days or until the adherent properties of the dressing are no longer adequate.
- Mepiform is waterproof and can be worn while bathing and showering.
- Mepiform has an ultraviolet protection factor (UPF) of class 5 in order to protect the scar.
- Mepiform is clinically proven to flatten, soften and fade red and raised hypertrophic and keloid scars. As with other scar treatment products, it may take at least 3 months to see an improvement of the scar, depending on the condition of the scar tissue.

Precautions

Should maceration or rash occur, remove the dressing and allow the skin to recover until the symptom has disappeared. Then continue treatment and gradually increase therapy time. If the symptom persists, discontinue use and consult a physician for advice.

Mepiform is latex-free. The sealant (glue) of the inner packaging contains natural rubber latex.

Do not use on open or infected wounds.

Product properties cannot be guaranteed if used after the expiry date.

Mepiform and Safetac are registered trademarks of Mölnlycke Health Care AB.

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Mepiform e Safetac são marcas registadas da Mölnlycke Health Care AB.