Mölnlycke®

# Mepore® Film & Pad

STERILE EO



CE 2797

Manufacturer

 Mölnlycke Health Care AB  
Gamlestadvägen 3C, Box 13080  
SE-402 52 Göteborg, Sweden

[www.molnlycke.com](http://www.molnlycke.com)

## Mepore® Film & Pad

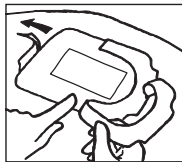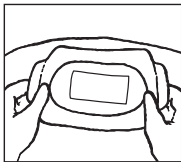Single use  
Usage unique

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[www.molnlycke.com/symbols](http://www.molnlycke.com/symbols)

Australian sponsor address:  
Mölnlycke Health Care Pty. Ltd.  
12 Narabang Way, Belrose, NSW 2085 AUSTRALIA

Toll free number :  
USA 1-800-882-4582  
Canada 1-800-494-5134

MD

- en Medical Device
- de Medizinprodukt
- fr Dispositif médical
- es Producto sanitario
- nl Medisch hulpmiddel
- sv Medicinteknisk produkt
- it Dispositivo medico
- fi Lääkinnällinen laite
- pt Dispositivo médico
- da Medicinsk udstyr
- el ιατροτεχνολογικό προϊόν
- pl Wyrób medyczny
- cs Zdravotnický prostředek
- hu Orvostechikai eszköz
- no Medisinsk utstyr
- sl Medicinski pripomoček
- bg Медицинско изделие
- ro Dispozitiv medical
- sk Zdravotnícka pomôcka
- tr Tibbi cihaz
- lt Medicinos priemonė
- lv Medicīniska ierīce
- et Meditsiiniseade
- ru Медицинское изделие
- hr Medicinski proizvod
- sr Medicinsko sredstvo
- is Lækningatæki
- ar جهاز طبي

- en Single sterile barrier system
- de Einzelnes steriles Barriersystem
- fr Système de barrière stérile unique
- es Sistema de barrera única estéril
- nl Enkelvoudig steriel barrièresysteem
- sv Sterilt singelbarriärsystem
- it Sistema di barriera sterile singola
- fi Yksinkertainen steriili sulkujärjestelmä
- pt Sistema único de esterilização de barreira
- da Sterilt enkeltbarrieresystem
- el Ενιαίο σύστημα αποστειρωμένων φραγμών
- pl System pojedynczej bariery sterylnej
- cs Systém jedné sterilní bariéry
- hu Egyszeres sterilgát-rendszer
- no Sterilt enkeltbarrieresystem
- sl Enojni sterilni pregradni sistem
- bg Единична стерилна бариерна система
- ro Sistem cu barieră sterilă simplă
- sk Systém s jednou sterilnou bariérou
- tr Tekli steril bariyer sistemi
- lt Vieno sterilaus barjero sistema
- lv Viena sterilās barjeras sistēma
- et Ühekordselt kasutatav steriilne barjäärisüsteem
- ru Одиночная стерильная барьерная система
- hr Jednokratni sustav sterilne barijere
- sr Jednokratni sistem sterilne barijere
- is Einfalt dauðhreinsað hindrunarkerfi
- ar نظام حاجز معقم أحادي

- en** Do not use if package is damaged
- de** Bei beschädigter Verp. verwerfen
- fr** Ne pas utiliser si l'emballage est endommagé
- es** No utilizar si el envase está dañado
- nl** Niet gebruiken indien verpakking beschadigd is
- sv** Skall inte användas om förpackningen är skadad
- it** Non utilizzare se la confezione è danneggiata
- fi** Älä käyttää jos pakkaus vaurioitunut
- pt** Não usar se embalagem danificada
- da** Brug ikke, hvis pakken er beskadiget
- el** Μην το χρησιμοποιήσετε αν η συσκευασία είναι κατεστραμμένη
- pl** Nie stosować, jeśli opakowanie jest uszkodzone
- cs** Nepoužívejte, je-li obal poškozen
- hu** Ne használja, ha a csomagolás sérült
- no** Må ikke brukes hvis forpakningen er skadet
- sl** Ne uporabite, če je ovoj poškodovan
- bg** Не използвайте, ако опаковката е увредена
- ro** Nu utilizați dacă ambalajul e deteriorat
- sk** Nepoužívajte, ak je obal poškodený
- tr** Ambalaj hasarlıysa kullanmayın
- lt** Nenaudoti, jei pakuotė pažeista
- lv** Nelietot, ja iepakojums ir bojāts
- et** Kahjustatud pakendi korral mitte kasutada
- ru** Не использ., если упак. поврежд.
- hr** Ne koristiti ako je pakiranje oštećeno
- sr** Ne koristiti ako je omot oštećen
- is** Notist ekki ef pakning er sködduð
- ar** لا يجوز استخدام الجهاز إذا كانت العبوة تالفة

- en** Wound pad size
- de** Größe Wundauflage
- fr** Taille de la matrice
- es** Medida del apósito para cubrir la herida
- nl** Formaat wondkussen
- sv** Sårdynans storlek
- it** Dimensione tampone
- fi** Haavatyynyn koko
- pt** Tamanho do penso para feridas
- da** Sårpudens størrelse
- el** Μέγεθος επιθέματος πληγής
- pl** Wielkość wkładki stykającej się z raną
- cs** Velikost krytí
- hu** Sebfező párna mérete
- no** Sårputens størrelse
- sl** Velikost obloge za rane
- bg** Размер на подложката за рани
- ro** Dimensiune tampon plagă
- sk** Veľkosť tampónu obväzu
- tr** Yara örtüsü boyutu
- lt** Žaizdos pagalvėlės dydis
- lv** Brūces spilventiņa izmērs
- et** Haavapadja suurus
- ru** Размер раневой прокладки
- hr** Veličina povoja za rane
- sr** Veličina jastučića za ranu
- is** Stærð grisju
- ar** مقياس ضمادة الجرح

## Mepore® Film & Pad

Absorbent adhesive film dressing

### Product description

Mepore Film & Pad is a sterile, transparent, self-adhesive, absorbent and shower proof film dressing with a bacterial barrier.

Mepore Film & Pad consists of:

- an absorbent wound pad with an integrated non-adherent wound contact layer.
- a shower proof, transparent polyurethane film coated with a solvent-based polyacrylic adhesive. The adhesive area is covered with a printed release liner.
- an application frame and tabs of siliconised paper.

Dressing material content: Acrylic adhesive, polyurethane film, viscose and polyester nonwoven

### Indications for use

Mepore Film & Pad is intended for use on low to moderately exuding wounds, such as surgical wounds, cuts and abrasions.

### Precautions

Do not use Mepore Film & Pad on patient with known hypersensitivity to acrylic adhesives or other materials/components of the product.

Take extra care when applying on fragile skin. Use on fragile skin may cause skin irritation.

Do not stretch the dressing when applying. Applying under tension may cause blisters.

If you see signs of infection e.g. fever or the wound or surrounding skin becoming red, warm or swollen, consult a health care professional for appropriate treatment.

Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.

Sterile. Do not use if sterile barrier is damaged or opened prior to use.

Do not re-sterilise.

### Instructions for use

Mepore Film & Pad can be used by lay persons under supervision of health care professionals.

1. Cleanse the wound in accordance to clinical practice. Dry the surrounding skin thoroughly. To ensure proper adhesion, do not use moisturizers on the skin covered by the dressing
2. Remove the printed release liner and apply the adhesive side to the skin without stretching the dressing. Ensure that the absorbent pad covers the wound.
3. Remove the paper frames and tear off the two white tabs. Gently press the border to ensure proper adhesion to the skin.
4. Upon removal grasp the edge and gently pull the film along the skin, in the direction of hair growth.

Care should be taken to ensure that Mepore Film & Pad is not applied under tension, to prevent shear forces causing damage to the skin. When applying over joints be careful to apply in such a way that it does not impair flexibility of the joint. After some surgical interventions excessive swelling may occur. In such case it could be better to use a more flexible dressing in order to minimise skin tension.

Mepore Film & Pad should be change before the dressing is fully saturated, at signs of leakage or as indicated by clinical practice.

Disposal should be handled according to local environmental procedures.

### Other information

If any serious incident has occurred in relation to the use of Mepore Film & Pad, you should report to Mölnlycke Health Care and to your local competent authority.

Mepore® is a registered trademark of Mölnlycke Health Care AB

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Mepore® er skráð vörumerki Mölnlycke Health Care AB