Mölnlycke Setopress

P.E.C MEDIUM / HIGH COMPRESSION BANDAGE
Setopress is a polyamide, elastane and cotton compression (medium/high) extensible ("PECCHE") bandage.

Indications for use
Setopress is for the treatment of venous leg ulcers.

Instructions for use
To ensure correct application, a simple visual guide is printed on both sides of the bandage. The visual guide consists of coloured rectangles which become squares when the correct extension is reached. On one side of the bandage are brown rectangles indicating high compression and on the other side are green rectangles indicating moderate compression.


N.B. You have a choice of compression levels. Brown rectangles give high compression. Green rectangles give moderate compression.

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
Extension +/-10%		Applied spirally with 50% overlap Limb circumference (cm/in)			
	<18cm	18-20cm 7-10.2in.	21-25cm 8.3-10.1in.	26-30cm 10.2-11.8in.	
Approx. sq. bandage pressure	70% (green square)	Not suitable	27-38 mmHg 1-1.5in.Hg	20-26 mmHg 0.8-1.0in.Hg	Not suitable
	100% (brown square)	Not suitable	35-50 mmHg 1.4-2in.Hg	27-36 mmHg 1-1.3in.Hg	19-26 mmHg 0.7-1in.Hg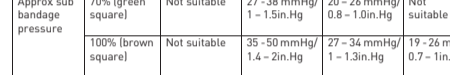


*A padding layer should be used prior to application

Extension +/-10%		Applied spirally with 50% overlap Limb circumference (cm/in)			
	<18cm	18-20cm 7-10.2in.	21-25cm 8.3-10.1in.	26-30cm 10.2-11.8in.	
Approx. sq. bandage pressure	70% (green square)	Not suitable	27-38 mmHg 1-1.5in.Hg	20-26 mmHg 0.8-1.0in.Hg	Not suitable
	100% (brown square)	Not suitable	35-50 mmHg 1.4-2in.Hg	27-36 mmHg 1-1.3in.Hg	19-26 mmHg 0.7-1in.Hg

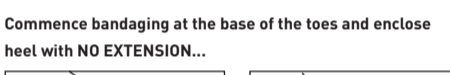
*A padding layer should be used prior to application

Commence bandaging at the base of the toes and enclose heel with NO EXTENSION...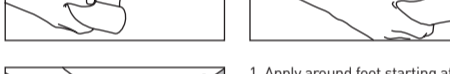

...NOW WITH EXTENSION continue up the leg in an ascending spiral

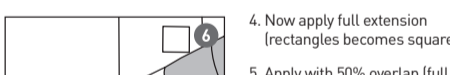...finally secure bandage just below the knee

Setopress may be applied over a primary wound dressing.

Contraindications
Do not use figure of eight bandaging. Do not apply if arterial insufficiency is present or suspected. Do not use on limbs with a circumference of less than 18cm/7in.

Precautions
Do not use Setopress on patient and/or user with known hypersensitivity to the ingoing materials/components of the product.

The effective circumference of the ankle when reading off the sizing table must take into account any padding layer. Setopress should only be applied by a trained bandager. Do not use bandage clips. This product must be used under the direction of a healthcare professional. Bandage should be changed in line with dressing changes. Discard and replace if soiled with bodily fluids.

Fibre content
Polyamide, Elastane, Cotton.

Special storage conditions and handling conditions
Disposal should be handled according to local environmental procedures.

Other information
If any serious incident has occurred in relation to the use of Setopress, it should be reported to Mölnlycke Health Care and to your local competent authority. The Mölnlycke and Setopress trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of companies. Size 10cm x 3.5m (relaxed roll)

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