

CAD Ref: PLSM129


Dimensions: 123 x 27 x 73

Mölnlycke®
 Ref no: 47642-02
 Created by: Natalie Whiteside
 Refer to ARS PD-530092 and Master Text IFU PD-572948

Colour information

- PMS 361
- PMS Cool Grey 10
- Cutter
- Varnish free

Mölnlycke Health Care, Gamlestadsvägen 3C, SE-402 52, Göteborg.

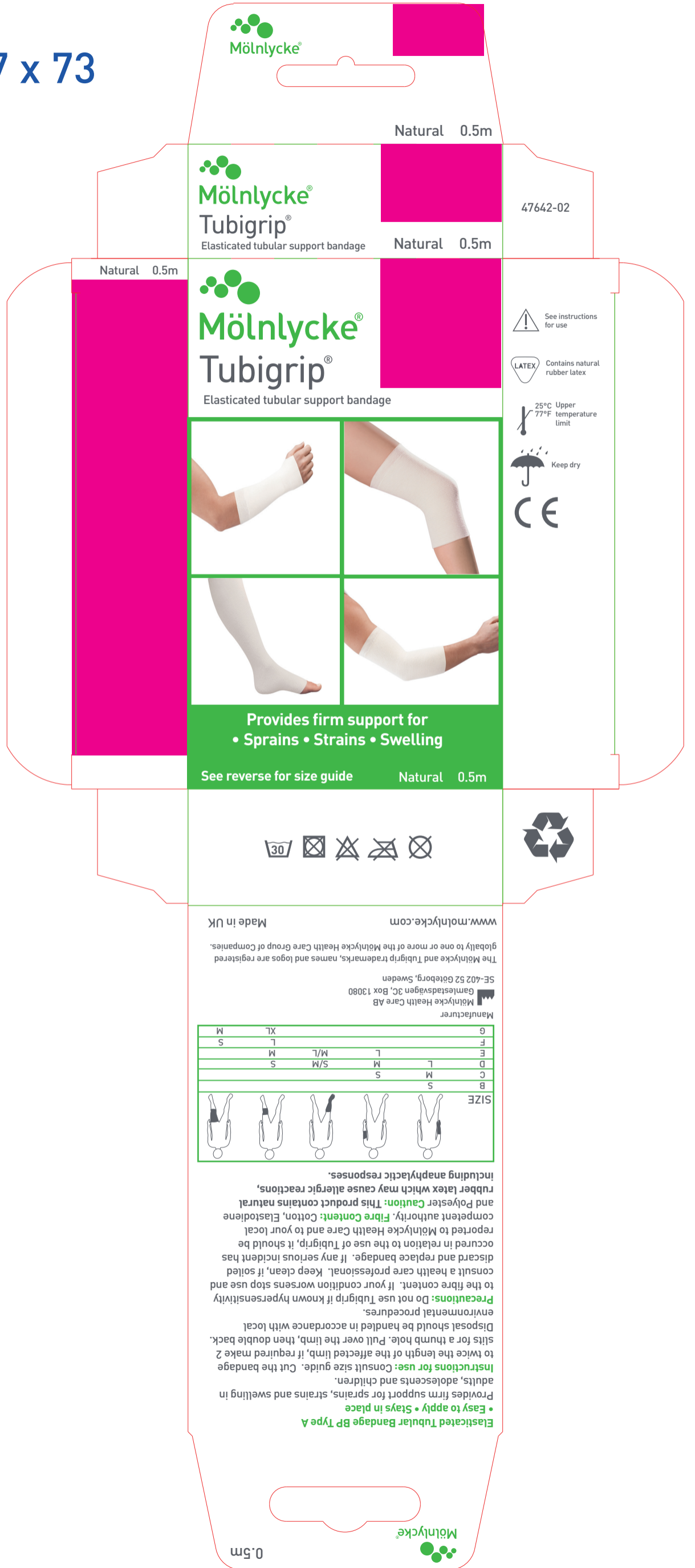

SIZE

B	S	S	S	S
C	S	M	S	S
D	S	L	M	S/M
E	S	L	M	M/L
F	L	L	M	M
G	XL	XL	XL	XL

Elasticated Tubular Bandage BP Type A
 • Easy to apply • Stays in place
 Provides firm support for sprains, strains and swelling in adults, adolescents and children.
Instructions for use: Consult size guide. Cut the bandage to twice the length of the affected limb, if required make 2 slits for a thumb hole. Pull over the limb, then double back. Disposal should be handled in accordance with local environmental procedures.
Precautions: Do not use Tubigrip if known hypersensitivity to the fibre content. If your condition worsens stop use and consult a health care professional. Keep clean, if soiled discard and replace bandage. If any serious incident has occurred in relation to the use of Tubigrip, it should be reported to Mölnlycke Health Care and to your local competent authority. **Fibre Content:** Cotton, Elastodiene and Polyester **Caution:** This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.

Manufacturer
 Mölnlycke Health Care AB
 Gamlestadsvägen 3C, Box 13080
 SE-402 52 Göteborg, Sweden
 The Mölnlycke and Tubigrip trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.
 www.molnlycke.com
 Made in UK



Mölnlycke®
Tubigrip®
 Elasticated tubular support bandage

Mölnlycke®
Tubigrip®
 Elasticated tubular support bandage

Mölnlycke®